

PUMP	MATTRESS
Power Supply: 120V/60Hz	Size: 36" (W) x 80" (L) x 8" (H)
Power: 6.4W	Air Cells Height (inflated): 8" (H)
Fuse Rating: 1A/250V	Air Cells Number: 20
Pressure Range: 30 ~ 60mmHg	Air Cell Material: Nylon/PVC
Dimensions: 11" x 6" x 5"	Cover Material: Nylon/PU wth Zipper
Cycle: 10 mins.	Weight Capacity(Max): 350lbs
Weight: 4.8 Lbs.	Weight: 16 Lbs.
Air Output: > 6L	CPR Emergency Valve
Normal and /Low Pressure Indicator Light	Low air loss cells & Quick Connector
Alternating and Static Mode Adjustable	Head cells with Pillow Function

MEDACURE LIMITED WARRANTY

This warranty is extended only to the original purchaser who purchases this product when new and unused from MedaCure or a dealer. This warranty is not extended to any other person or entity and is not transferable or assignable to any subsequent purchaser or owner. Coverage under this warranty will end upon any such subsequent sale or other transfer of title to any other person

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

MedaCure warrants the mattress and cover when purchased new and unused to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from MedaCure or a dealer, with a copy of the seller's invoice required for coverage under this warranty. MedaCure warrants the control unit when purchased new and unused to be free from defects in materials and workmanship for a period of one (1) year from the date of purchase from MedaCure or a dealer, with a copy of the seller's invoice required for coverage under this warranty. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at MedaCure's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. MedaCure's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your MedaCure product or MedaCure customer service only if purchased from MedaCure direct. Provide model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

DO NOT return products to MedaCure without prior consent. Please obtain a return authorization number from MedaCure. C.O.D. shipments will be refused; return shipping charges must be prepaid. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty days of return authorization date.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE INSTRUCTION MANUAL THAT COMES WITH THE PRODUCT. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECT TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT MEDACURE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF MEDACURE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND MEDACURE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN MEDACURE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY MEDACURE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. MEDACURE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS

User Manual

MODEL # OS200-C

**8" LOW AIR LOSS MATTRESS
with ALTERNATING PRESSURE**



Class I Medical Device according to Directive 93/42/EEC and further modifications

MODEL # & DESCRIPTION

MODEL # OS200-C Mattress and pump with pressure adjustment.

INTRODUCTION

Thank you for purchasing the OASIS Model # OS200-C Mattress with Alternating Pressure and Low Air Loss by MedaCure. This user's manual provides suggestions on how to correctly use the product and gives valuable advice regarding your safety. Please read through the manual carefully before using the product. Should you have any questions, please contact your dealer for advice and assistance.

INTENDED USE

The OASIS mattress is designed for bed sore and wound care therapy treatment and prevention, which may occur during an extended hospital stay and nursing home /long term care environment.

CAUTION!

- Do not use the product for any purpose not indicated in this manual.
- Only qualified personnel trained in the treatment and prevention of bedsores should operate this device.
- MedaCure Inc. declines all liability for any consequences resulting from incorrect use of this product and from unauthorized alteration to the frame of the product.
- The manufacturer reserves the right to change the information contained in this document without prior notice

STANDARDS

The system has been tested and successfully approved to the following standards:
EN 60601-1
EN 60601-1-2

GENERAL WARNINGS

Keep this manual in a safe place and refer to the manual before use and under proper medical supervision. Improper operation of this system may cause damage to the product and possible injury to the user.

1. DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, please contact a healthcare professional, dealer or authorized technician before attempting to use this equipment, otherwise injury or damage may occur.
2. Consult with physician or therapist to determine the correct adjustment and the correct use of this device.
3. Keep the packed kit away from heat sources.
4. SERVICE LIFE- the device use limit is defined by the wear of the parts.
5. DO NOT allow children to play with or operate the pump.

INSTALLATION

1. Place the mattress on the bed frame with the air hose connectors facing toward the footboard.
2. Hang the pump with the hooks on back of the pump on the footboard or place on a flat surface.
3. Connect the air hose connectors from the air mattress to the pump unit. Ensure the air hoses are not kinked or tucked under mattress.
4. Plug the power cord into the electrical outlet.
5. Turn the switch ON.

WARNING!

- Make sure the pump unit is suitable for the local power voltage.
- Do not position the equipment so that it is difficult to disconnect the plug.

BEFORE USE

Check and confirm that the unit is assembled correctly. Pay particular attention to the air hose connector.

USE WARNINGS

1. Electric shock - Avoid electric shock
It is very dangerous to use electric products improperly or damaged ones.
 - 1) Unplug the pump control unit if you will not use it for a long time.
 - 2) Do not use the pump control unit near water or any other fluid.
 - 3) Avoid having the cable, plug and main control unit being in contact with water or any other fluid.
 - 4) Unplug the pump control unit if or when it is in contact with water or other fluid.
- Do not open the control unit by yourself. This should only be done by an authorized technician.

Safe Place

- 1) Avoid damaging the plug or cables when moving the patient or bed.
- 2) Ensure that the cables are safe and tidy and not caught in the bed mechanism or resting on the floor.
- 3) Keep sharp objects or strong acid and alkali chemical agents away from the mattress.
- 4) Fuse will blow to protect the device when electric short circuit is caused by the current loading.

Cautions - Keep the system away from fire, avoid electric shock or any danger that will cause injury to the user.

- 1) Only use the system under the supervision of a nurse or care giver that understands how to operate this device.
- 2) Do not use any parts that did not come with this unit or supplied by MedaCure or an authorized dealer.
- 3) Ensure that the mattress, cover, cables and power outlet are not in contact with fire, heat or strong acid and alkali chemical agents.
- 4) Do not modify the pump and mattress without authorization of the manufacturer and do not insert objects in to the control unit or mattress.
- 5) Please plug the control unit into the ground power supply.
- 6) Do not use the product in case of the following:
 1. the plug or the cable is damaged.
 2. the system sounds abnormal or vibrates intensely.
 3. the control unit fell on the floor and caused damaged to the housing
 4. the control unit fell into water or other fluid.

NOTE

In case the pressure is constantly low, check for any holes on the interchangeable air cells or on the connection tubes. Check also that the CPR valve is firmly connected.
If necessary, contact your dealer to exchange the air cells or connection tubes/hoses.

STORAGE

- 1) Switch off power from main and unplug, separate the pump from the mattress and place the mattress flat.
- 2) Fold the mattress in two from mattress foot to head.
- 3) Allow time for air to escape from mattress to reduce storage space.
- 4) Close the protective cap cover on the quick connector.

Maintenance

In case of repair, use only original spare parts and accessories.

CLEANING AND DISINFECTION


WARNING!

Unplug the pump before starting the cleaning procedure.

1. Clean the mattress with mild soap.
2. Do not immerse or soak pump unit.
3. After cleaning, dry the mattress without direct sunlight exposure.
4. Check that the connection tubes are not obstructed by dust or other object. Use a soft swab to clean the tubes if necessary.

NOTE Never use acids, alkalis or solvents such as acetone to clean the mattress.

Product Functions:

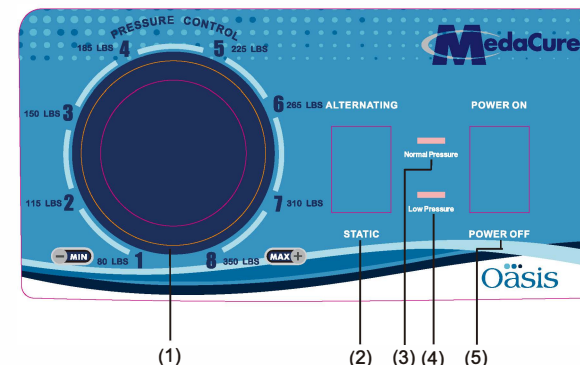

- (1) Pressure Adjust Valve: Adjust the valve to increase or decrease the pressure for a softer or firmer setting between 1-8 range or by patients weight as noted on the panel.
- (2) Alternating/Static Mode Switch: Press this switch to adjust the mode from Alternating to Static.
- (3) Normal Pressure Indication Light: When the pressure is in normal operation, the light will turn on.
- (4) Low Pressure Indication Light: When the product is abnormal, the low pressure indication light will turn on.
- (5) Power Switch: Press this button to turn on/off the pump.