

## Adjusting The Hand Brake System


### **\*\* Safety Notice \*\***

The rollator's hand brake system complies with the quality standards set by the manufacturer. This system has been positioned for the best performance settings. To find the setting that suits you personally, follow the instructions below. The hand brake systems may need occasional adjustment due to wear from normal use.

Hold firmly in place do not turn.

To tighten the hand brake adjustment, turn the nut clockwise.

To loosen the hand brake, adjustment, turn the nut counter clockwise.

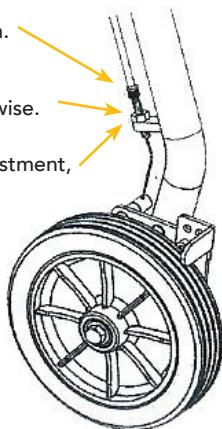

## Maintenance

MedaCure recommends that users inspect this product periodically to confirm the security of parts and hardware and make sure they work properly and are not worn, torn, frayed, or loose, as well as checking for obstructions or impediments to normal, safe operation. Do not use the product if any part or mechanism appears to be unsafe or compromised. Please contact the product seller or an authorized dealer for warranty or service

## Medacure Limited Warranty

The MedaCure Bariatric Rollator is guaranteed to be defect-free in materials and workmanship for one (1) year from the date of purchase.

Damages that are not covered by this warranty include defects due to misuse, abuse or negligence and normal wear and tear on the device. This warranty does not include or extend to non-durable components, (i.e., rubber accessories, casters, and grips), as they need periodic replacement due to normal wear.

For questions about your MedaCure device and/or the warranty, feel free to contact an authorized MedaCure dealer.



Innovative Healthcare Solutions

© 2020 MedaCure Inc. All rights reserved.  
MedaCure is a registered trademark of Medacure Inc.  
East Orange, NJ 07017. Made in China.

## Bariatric Rollator

Weight Capacity: 500 lbs.

Model # RB500



Innovative Healthcare Solutions

MedaCure Inc.  
545 N. Arlington Avenue  
East Orange, NJ 07017, USA  
[www.medacureinc.com](http://www.medacureinc.com)

## Assembly Instructions

**CAUTION:** Please read all instructions in this manual carefully before attempting to adjust or operate the rollator.

1. Remove backrest, handle with tube, Rollator frame and Basket from the box.
2. Open frame and slide padded backrest into frame brackets, taking care that snap bottoms extend through holes.
3. Remove tightening knobs from the packaging and insert handle tube into frame at desired height. Line up holes accordingly. Slide bolts through holes from the inside and screw. Tighten knob until it is secure.



## Safety Precautions

1. The Rollator must be fully opened, set down, and the folding bar under seat locked in the down position.
2. Both knobs must be fully tightened and at equal heights. Test Rollator for stability before applying full weight to hand grips.
3. Test the loop locks before using the Rollator. Lock bar on rear wheels should prevent wheels from rolling when wheels are locked (see illustrations for locking and releasing locks below). Please take unit back to dealer for adjustment if the wheels roll while brakes are locked. Wheels should roll freely when locks are released.
4. Loop locks should always be locked before sitting. Do not use seat when unit is on an incline or uneven ground.
5. Tires should be replaced when showing signs of significantly wear or damage.
6. No one should push Rollator while someone is seated on it. The Rollator is a walking aid only and is not to be used as a transportation device.
7. Weight Capacity is 500 lbs / 226 kg.

## Operating the Loop Lock

Always engage and disengage wheel locks on both sides simultaneously.

1. To Slow Down or Come to a Temporary Stop — Pull up slowly on the loop locks with four fingers until unit slows down or comes to a temporary stop.
2. To Temporarily Lock Wheels in Position — Press down on both loop locks with palms until you hear a “click” that will tell you wheels are locked.
3. To Release Wheel Locks — Pull up on both loop locks with palms of hand to original position.

