



Innovative Healthcare Solutions

www.medacureinc.com

**Rolling Shower Wheel Chair
with Detachable Commode**



Innovative Healthcare Solutions





MODEL # SWC1816




www.medacureinc.com

Assembly Instructions



1. Remove the frame, backrest set, and headrest from carton
2. Insert the front casters into the frame and use "E" clips to secure them in place.
3. Insert the quick release pins into the middle of the large wheelchair wheels and slide wheels into the sides of the frame. The wheel will lock into place.

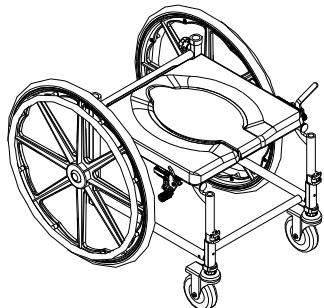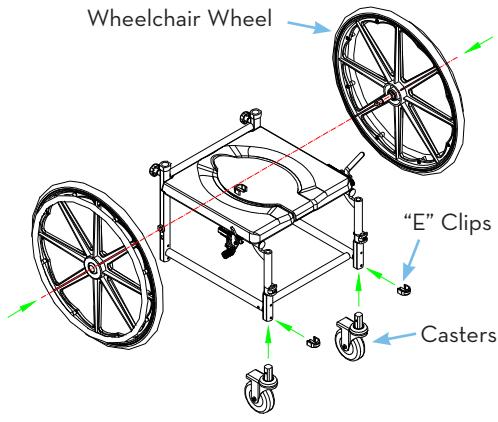


4. Assemble the footrests by inserting the footrests into the top of the front legs from the outside of the frame. Once inserted, turn the footrests inward until they lock in position.

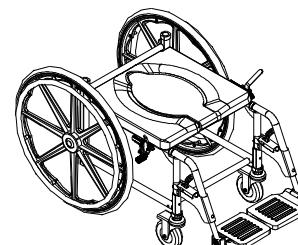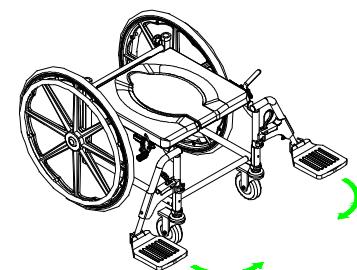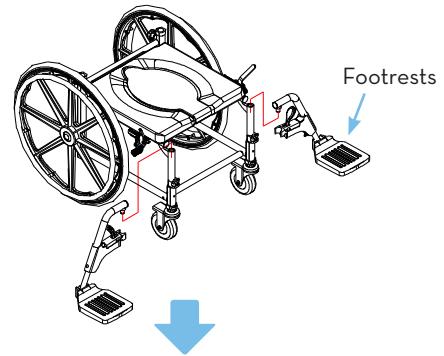

- Assemble the backrest by inserting the supporting poles into the back of seat frame. Secure in place using the lock knobs provided.

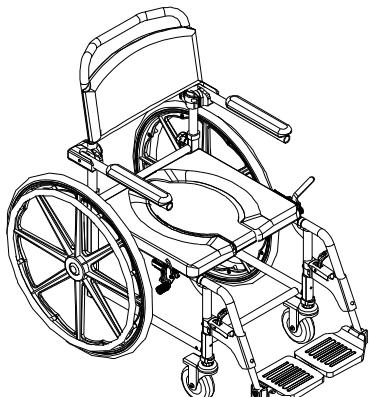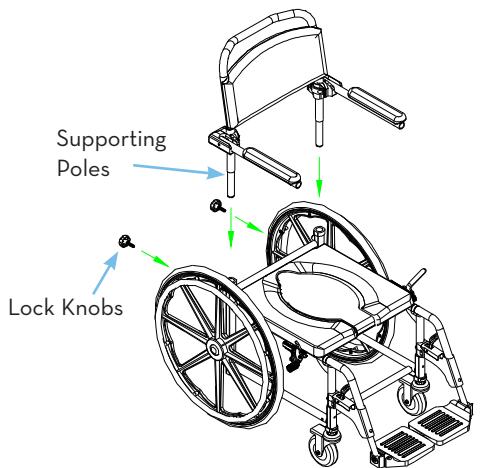

- The angle of footplate is adjustable to fit the user. Using the allen key, loosen the screws by turning counter-clockwise, adjust the footplate to a suitable angle of footplate and re-tighten.

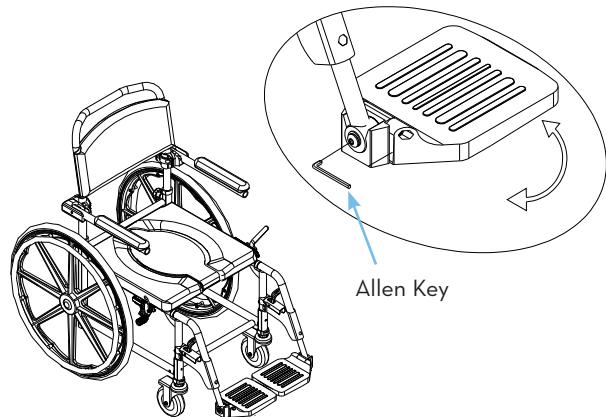

- Slide commode bucket underneath seat into receptacle.
- Remove cover over commode bucket before use.

CAUTIONS

1. Toilet seat must be in down position before use. Failure to do so may result in damage to commode and possible injury.
2. Do NOT install commode without first reading and understanding this assembly and instruction sheet. If you can not understand these instructions, contact a authorized Medacure representative or dealer before attempting to install.
3. Inspect all parts for any damage before assembly. If there is any damage - DO NOT USE. Contact an authorized dealer for further instructions.
4. Ensure that the "E" Clips protrude through the hole of each leg extension. This will ensure that the leg extensions are securely locked in position.
5. Ensure ALL screws, nuts and/or bolts are tightened.
6. Footrests can be folded up when not in use by folding the foot rests upward, releasing the locking mechanism and folding inward to the front of the chair.
7. Users with limited physical strength should be supervised or assisted while using commode.
8. Use the arm supports only for assistance. DO NOT attempt to use the arms to support full body weight.
9. Arms should be pushed back to assist in transfer onto/off the chair.

**WEIGHT CAPACITY 275 LBS
SEAT DIMENSIONS: 18" W X 16" D**

MEDACURE LIMITED WARRANTY

This product is warranted to be free of defects in materials and workmanship for three (3) years from the date of purchase

For warranty service, please contact the DEALER from whom you purchased this product. **DO NOT return product to MedaCure without prior consent.** Provide model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about this device or warranty, please contact an authorized Medacure dealer or representative.



Innovative Healthcare Solutions

© 2020 Medacure Inc. All rights reserved.

Medacure is a registered trademark of Medacure Inc. East Orange, NJ 07017 USA

Made in China