



WINGS



WINGS

**HEAVY DUTY
BARIATRIC WHEELCHAIR**



MODEL # WC2620 (26" W X 20" D)

MODEL # WC2820 (28" W X 20" D)

MODEL # WC3020 (30" W X 20" D)

PLEASE REFER TO LABEL ON CROSSBAR FOR CORRECT MODEL NUMBER



Innovative Healthcare Solutions

www.medacureinc.com



Innovative Healthcare Solutions

www.medacureinc.com

Please read this manual carefully for best safety measures and proper usage and maintenance.

USE AND SAFETY

WHEEL LOCKS - Adjust both wheel locks to the brake position before transferring in or out of the wheelchair, leaning forward in chair or while in an elevator or on a wheelchair lift.

Proper adjustment of wheel lock is at least 1/8" when in the locked position. Locks are employed by pushing the brake handle completely forward. Adjustments to locks are made by loosening or tightening the nut on the carriage bolt. Embed lock shoe at least 1/8" by sliding clamp toward rear wheel while handle is employed in locked position. Tighten nut and bolt to secure in position. Confirm the lock is employed correctly before actual use.

TIRES - Solid rubber tires need minimal maintenance, but should be cleaned occasionally with a damp cloth. Replace if tires become severely worn or cracking appears.

FRONT CASTERS - Casters should be checked periodically and lubricated when necessary. Lubricate the casters by removing the fork and greasing the caster stem bearings. Lubricate wheel bearings by removing the wheel from the fork. Reassemble fork and wheel by tightening nuts so that it's not too tight or loose and the wheels roll easily.

REMOVAL OF SWING-AWAY FOOT RESTS/LEG RESTS - Release lock by pushing on the front rigging lever. Swing-away foot rest /leg rest out and sideways and lift to remove.

ADJUSTING LENGTH OF FOOTREST/LEG REST - When adjusting the length of the foot rests or leg rests, you should sit in the wheelchair with your feet on the the foot plates. Loosen bolt on the clamp around support assembly. Raise or lower the foot plate to a position that provides maximum support of your thighs. Front rigging should not be so low that it will create uncomfortable pressure to thighs at front of the seat. To ensure proper clearance the footrest should not be lower than 2-1/2" from the ground.

ADJUSTING/ELEVATING LEG REST - Lift the foot plate on the leg rest assembly to desired position. When you release the leg rest assembly it will automatically lock into place. To lower, push out on leg rest lever and lower foot plate to desired position.

REAR WHEELS - The rear wheels should be inspected at least once a month to ensure proper tightness. If wheel shakes or is loose, release lock nut and tighten axle bolt slightly. Test for proper tightness by leaning the wheelchair to one side and spin the adjusted wheel.

GENERAL CARE - Wipe chrome parts with a clean, soft cloth at least once a week. If exposed to moisture, dry immediately. Do not use an abrasive cleanser that will scratch the finish.

TO OPEN WHEELCHAIR - Face the front of the wheelchair and place your hands on the seat rails. Push down and out until the seat is completely open and level.

TO CLOSE/FOLD WHEELCHAIR - First fold foot the plates to a vertical position. Grab the seat with both hands at front and back edge and lift. Or, tilt the wheelchair to one side and lift up on seat rail.

REMOVABLE ARMS - Remove arms by depressing spring button and lifting arms out of the front and rear sockets.

CAUTIONS AND SAFETY

STABILITY – The Wings Wheelchair is designed to be safe and stable as long as center of gravity is maintained while going about normal activities. Care must be taken while performing activities that shift your center of gravity in the chair. Never reach so far that require you to shift in the seat or bend over out of the chair. When you have to reach for objects that require you to lean, be certain front casters are pointing in a forward position that extends the wheel base and will stabilize the chair.

RAMPS - Be sure of your own capabilities, limitations and strength before attempting to go up a ramp. Correct techniques on ramp negotiation should be shown to you by a qualified professional. Assistance is recommended when going up or down inclines greater than 10°. If you must stop on an incline, avoid sudden, abrupt movement when you resume movement. Because of backward angle of the chair, sudden abrupt movement could cause instability. Attempting to use the wheel lock brakes could result in accidental locking, tipping to one side, or cause a sudden stop. Avoid changing direction when going down an incline, as this could result in instability of tipping over.

WEIGHT CAPACITY: 700 LBS

**SEAT DIMENSIONS:
26", 28", 30" W X 20" D**

REFER TO LABEL ON CROSSBAR FOR RELATED MODEL NUMBER

MEDACURE LIMITED WARRANTY

This product is warranted to be free of defects in materials and workmanship for three (3) years from the date of purchase

For warranty service, please contact the DEALER from whom you purchased this product. **DO NOT return product to MedaCure without prior consent.** Provide model number, and the date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about this device or warranty, please contact an authorized Medacure dealer or representative.



Innovative Healthcare Solutions

© 2020 Medacure Inc. All rights reserved.
Medacure is a registered trademark of
Medacure Inc. East Orange, NJ 07017 USA
Made in China